


water for injections

Available here

safe snap amp snapper

1. Place the ampoule in the safe snap so it is vertical and with **the blue dot facing out**.
2. Close the bottom section.
3. **Tap the top of the ampoule** to knock the water out.
4. Close and **snap**.
5. **A catch on the back locks the safe snap in the open position** to provide a stable base to draw up from.
6. **Draw up.**
7. **Empty the used amp into a sharps bin straight from the safe snap** – there's no need to touch the glass.
8. **After use, throw away any unused water.**

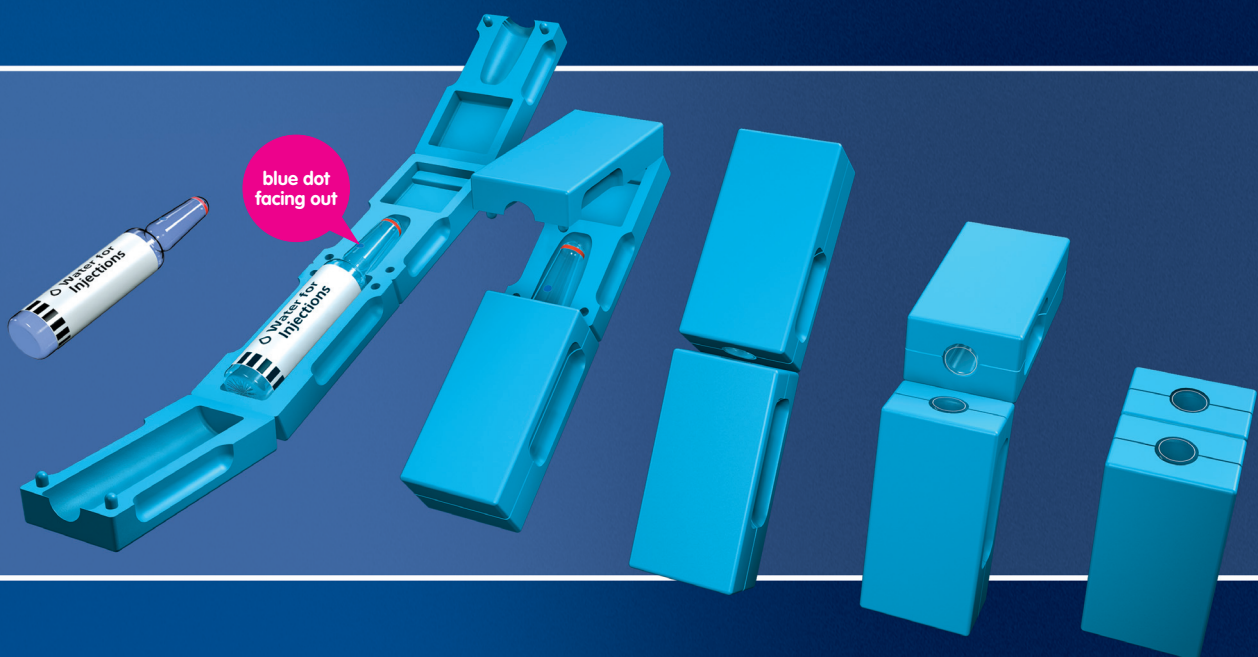

**EXCHANGE
SUPPLIES**

Exchange Supplies, 1 Great Western Industrial Centre, Dorchester, Dorset DT1 1RD
Tel: 01305 262244 Fax: 01305 262255 Email: info@exchangesupplies.org

exchangesupplies.org